QUICK REFERENCE GUIDE HAND CONTROL

NOTE: This quick reference guide offers basic information on only hand control components and functions. Read the owner's manual and all other supplemental information thoroughly before operating the chair for information on safety, basic operation, and care and maintenance.

The illustrations used reflect those hand controls with the highest level of functionality. The hand control you receive may have fewer options, but basic functionality is the same.

The hand control on your chair may be equipped with an LED that indicates power to the hand control. If this LED is present and not lit when the hand control is in use, contact your authorized Provider.

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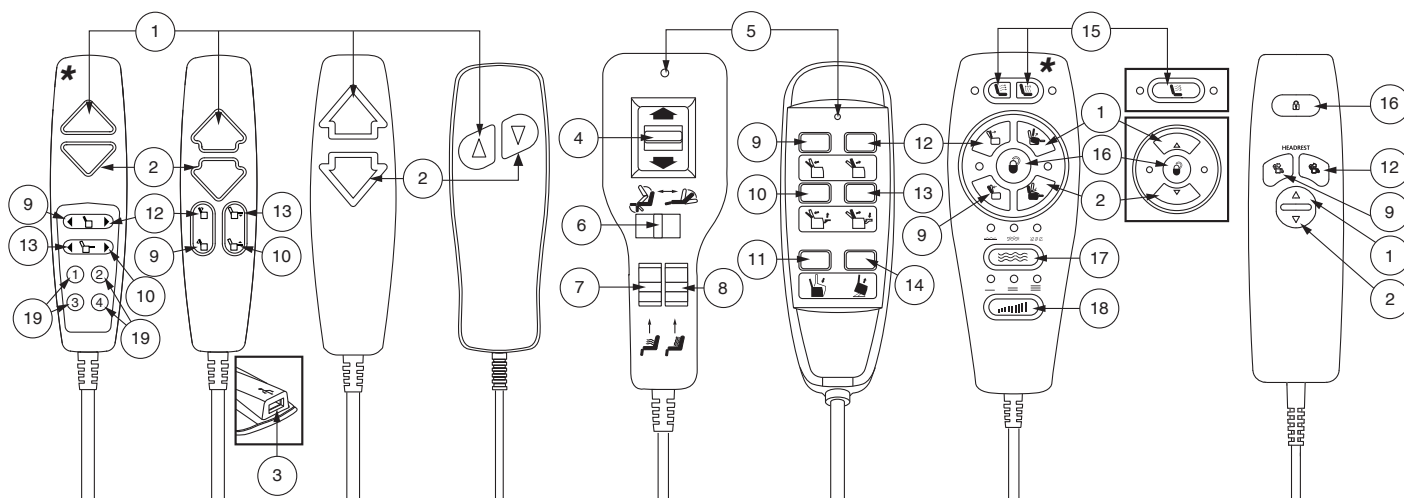
Rev M/September 2024

1 UP BUTTON
Press and hold to raise chair to seated or standing position.

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2 DOWN BUTTON
Press and hold to lower chair to seated or reclined position.

NOTE: On infinite position models, only the footrest is extended into a reclined position when this button is pressed. The backrest remains upright.

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3 USB PORT
Use for USB compatible devices.

NOTE: iPhone users must use ONLY MFI-certified cables when charging iPhone devices.

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WARNING! In the event of a power outage do NOT use the USB port to charge any device. Doing so may deplete the back up battery, which may cause your chair not to function as expected.

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4 UP/DOWN SWITCH
Controls seat lift and recline positions. If equipped with INHIBIT SWITCH, also controls footrest.

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5 POWER LED
Indicates power to the hand control. Will only light when the hand control is in use (any button pressed to adjust chair).

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6 INHIBIT SWITCH
If switch is set to the left, the UP/DOWN SWITCH controls the lift position of the chair and footrest. If switch is set to the right, the UP/DOWN SWITCH controls the recline position of the seatback.

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7 HEAT SWITCH
Slide to the up position to activate the heat option. Slide to the down position to deactivate the heat option. Will shut off automatically after 20 minutes.

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8 MASSAGE SWITCH
Slide to the up position to activate the massage option. Slide to the down position to deactivate the massage option.

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9 RECLINE BUTTON
Press and hold to recline the seatback.

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10 FOOTREST UP BUTTON
Press and hold to raise the footrest.

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11 SIT BUTTON
Press and hold to lower chair to seated position.

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12 INCLINE BUTTON
Press and hold to raise the seatback.

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13 FOOTREST DOWN BUTTON
Press and hold to lower the footrest.

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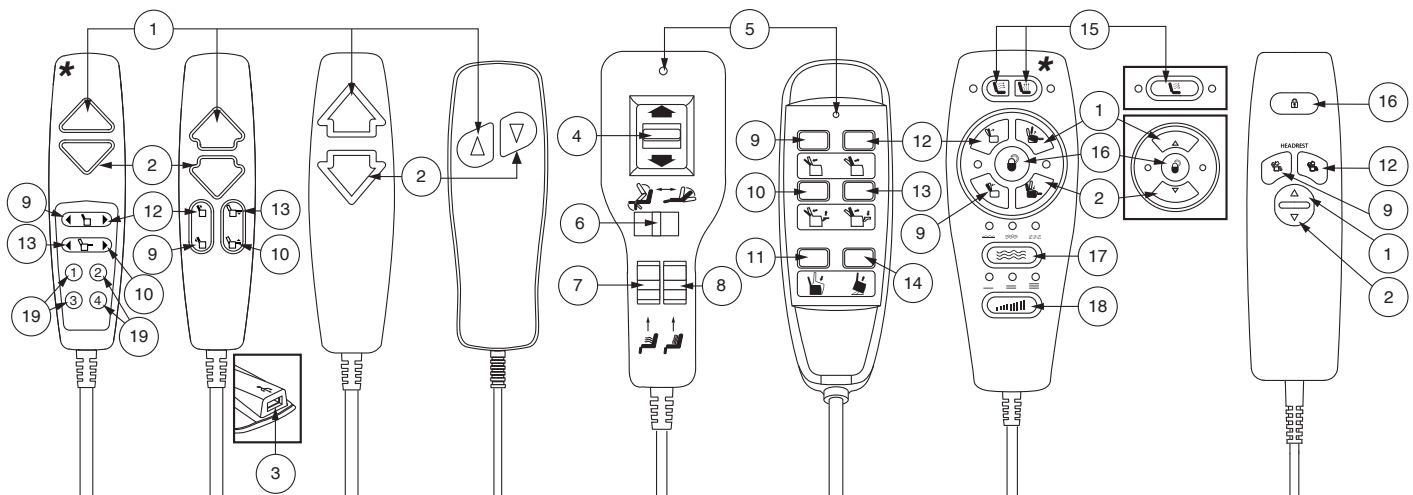
14 STAND BUTTON
Press and hold to raise chair to standing position.

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15 HEAT BUTTON (SEATBACK) AND (SEATBASE)
 Push once (1x) for low heat—green LED.
 Push again (2x) for high heat—red LED.
 Push again (3x) to shut off—no LED.
 Will shut off automatically after 20 minutes.

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16 LOCK/UNLOCK BUTTON
 Press and hold the Lock/Unlock button for approximately 5 seconds to lock the hand control, the Lock/Unlock button will illuminate when locked. To unlock, press and hold the Lock/Unlock button for approximately 10 seconds, the hand control buttons/LEDs will all flash indicating the hand control has been unlocked and the Lock/Unlock Button will no longer be lit.

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17 MASSAGE BUTTON WITH MODE
 Push once to activate massage option. Push again to cycle through modes indicated by LEDs above button (wave, steady, or pulse). Press and hold to shut off massage. Will shut off automatically after 20 minutes.

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18 INTENSITY BUTTON
 Push to cycle through the 3 intensities for the massage option indicated by the LEDs above button (low, medium, and high).

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19 MEMORY BUTTONS
 Press and hold Memory Button 1 and Memory Button 2 for three (3) seconds. The LEDs on the memory buttons will flash to indicate that the hand control is in "set" mode. Press and hold the desired memory button for three (3) seconds to set the memory position. The LEDs will stop flashing.
 To go to a preset memory position, press and hold the desired memory button.

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***NOTE: For hand controls equipped with memory buttons, flash codes are not applicable. However, the USB port will still become disabled once the chair enters low power mode. If your hand control is equipped with memory buttons and the USB port stops working, contact your authorized Pride Provider for assistance.**

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